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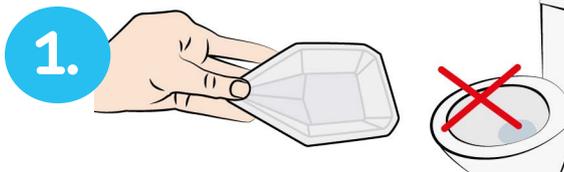
Home testing for chlamydia and gonorrhoea from urine

PLEASE NOTE BEFORE COLLECTING THE SAMPLE: The sample should be collected no earlier than 5 days after having unprotected sex. Collect the sample at least two hours after your last urination. Do not wash the urinary meatus before sample collection.

CONTENTS OF THE TEST KIT: Sample container, pipette and sample tube, plastic bag (incl. absorbent pad), fillable form with a barcode label.

The sample tube contains colourless liquid. This liquid contains harmful substances: Do not touch the inner parts of the sample tube/cap or make contact with the liquid. If any liquid splashes on your skin, rinse the area with plenty of water.

1. Take the sample container with spout from the test kit. **It is important that you collect the first part of the urine stream.** Start urinating into the sample container and fill it about half full. After that, you can continue urinating into the toilet as usual.
2. Take the pipette and the sample tube with a cap from the kit. Open the cap of the sample tube and use the pipette to transfer urine from the sample container into the sample tube. Keep transferring urine to the sample tube until the **surface of the urine reaches between the lines marked on the side of the sample tube (indicated by a green arrow in the image).** Do not fill the sample tube completely. Close the cap tightly.
3. Fill out the date and time of your sample collection in the form. Remove the label on the form and stick it on the side of the sample tube so that the barcode is vertical to the tube.
4. Put the sample tube with the absorbent pad inside the plastic bag included in the test kit and close the bag. Put the bag and the completed form into the test kit.
5. Close the test kit carefully and make sure that it stays firmly closed with a tape or the label included in it.
6. Write your name and address on the section titled "Sender" on the test kit.
7. **Mail the test kit**, the postage has been paid. You will receive the test results primarily in the Fimlab mobile app (read more: fimlab.fi/mobiilisovellus) or to your mobile phone number as a text message. The results will also be delivered to the patient information systems in your welfare area and MyKanta.

You can learn more at fimlab.fi/en/services/remote-testing-for-stds or by contacting Fimlab's customer advice centre: tel. 03 3117 4445 (weekdays 8–16).