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Home testing for chlamydia and gonorrhoea: Vaginal swab test

PLEASE NOTE BEFORE COLLECTING THE SAMPLE: The sample should be collected no earlier than 5 days after having unprotected sex.

CONTENTS OF THE TEST KIT: There are two swabs in the kit – use **only the cotton swab** to collect the sample.

The sample tube contains colourless liquid. This liquid contains harmful substances: Do not touch the inner parts of the sample tube/cap or make contact with the liquid. If any liquid splashes on your skin, rinse the area with plenty of water.

1. Take the **cotton swab** from the test kit. Do not touch the soft end of the cotton swab (i.e. the end that is used for sample collection). Hold the cotton swab in your hand below the cut-off point marked on the stem of the swab. Using your other hand, spread the labia around your vaginal opening.
2. Insert the cotton swab into the vagina to a depth of about 5 cm. Rotate the cotton swab inside the vagina for about 30 seconds, making sure that it is rubbing the vaginal walls. Gently pull out the cotton swab. **Do not place the swab anywhere or touch anything with it.**
3. Keep holding the swab in your hand and open the sample tube cap. Insert the cotton swab into the sample tube until **the cut-off point on the stem and the edge of the sample tube are aligned**. **Make sure that the end of the swab makes contact with the liquid in the tube only after the stem has been cut off.**
4. Carefully cut off the cotton swab at the cut-off point (bend it against the edge of the sample tube). Leave the tip of the cotton swab that you used to collect the sample in the sample tube. Close the cap of the sample tube tightly.
5. Fill out the date and time of your sample collection in the form. Remove the label on the form and stick it on the side of the sample tube so that the barcode is vertical to the tube.
6. Put the sample tube and the absorbent pad inside the plastic bag included in the test kit and close the bag. Put the bag and the completed form in the test kit box. Close the test kit box carefully: make sure that it stays firmly closed with a tape or with the sticker included. Write your name and address on the "Sender" field on the test kit box.
7. **Mail the test kit box**, the postage has been paid. You will receive the test results primarily in the Fimlab mobile app (read more: fimlab.fi/mobiilisovellus) or to your mobile phone number as a text message. The results will also be delivered to the patient information system in your wellbeing services county and MyKanta.

You can learn more at fimlab.fi/en/services/remote-testing-for-stds
or by contacting Fimlab's customer advice centre: tel. 03 3117 4445 (weekdays 8–16).

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tai Fimlab Laboratoriot Oy:n asiakasneuvonnasta puh. 03 3117 4445 arkisin klo 8–16.