

CardioSoft:

MUSE-SIIRRON ESTÄMINEN KATKON AJAKSI

1. Klikkaa System Configuration-painiketta alkunäytöltä

2. Siirry MUSE-välilehdelle

3. Poista ruksi kohdasta
'Request MUSE Data'

4. Laita täppä kohtaan
'No data Transfer to MUSE'

5. Paina OK

The screenshot shows the 'System Configuration' dialog box with the 'MUSE' tab selected. The 'Setup for MUSE' section has the 'Request MUSE Data' checkbox unchecked. The 'Store procedure for MUSE' section has the 'No data transfer to MUSE' radio button selected. The 'Data Transfer to MUSE via Shared Directory' section is also visible, with fields for Shared Directory, Directory User Name, and Directory Password. The 'Patient ID Length Limit' and 'Limit Text Entry to MUSE Length' checkboxes are also present.

MUSE-SIIRRON SALLIMINEN KATKON JÄLKEEN

1. Laita ruksi kohtaan
'Request MUSE Data'

2. Laita täppä kohtaan
'Data Transfer to MUSE via
Shared Directory'

3. Paina OK

The screenshot shows the 'System Configuration' dialog box with the 'MUSE' tab selected. The 'Setup for MUSE' section has the 'Request MUSE Data' checkbox checked. The 'Store procedure for MUSE' section has the 'Data Transfer to MUSE via Shared Directory' radio button selected. The 'OK' button is highlighted at the bottom right.

Section	Field/Option	Value
Setup for MUSE	<input checked="" type="checkbox"/> Request MUSE Data	
	MUSE Site	2
	MUSE Web Server	museserver
	MUSE User Name	webuser
	MUSE Password	*****
	<input type="checkbox"/> SSL Connection	
Store procedure for MUSE	<input type="radio"/> No data transfer to MUSE	
	<input type="radio"/> Save MUSE data to medium	
	<input type="radio"/> Data transfer to MUSE via the network	
	<input checked="" type="radio"/> Data Transfer to MUSE via Shared Directory	
	<input type="checkbox"/> Delete local test data after transfer to MUSE	
Location Number	1	
Cart/Device Number	0	
MUSE Software Version	5C and above	
Patient ID Length Limit	11	
Limit Text Entry to MUSE Length	<input type="checkbox"/>	